IN SAFE HANDS

Reflexx S.p.A., Via Passeri 2, 46019 Viadana (MN) | ITALY

DISPOSABLE EXAMINATION NITRILE GLOVE

Reflexx N350
USER INFORMATION SHEET
rev. UIS 04 feb. 2022

CE 2797


PPE cat. III Reg. EU 2016/425 EN ISO 374-1:2016+A1:2018 TYPE B, EN ISO 374-5:2016 VIRUS (ISO 16604:2004) EN ISO 21420:2020

Table with 4 columns: Chemical compound, OOD, Performance level, Breakthrough time, EN374-4:2018, and % degradation. Lists various chemical tests like Sodium Hydroxide 40%, Sodium Hypochlorite 10-13%, etc.

Module D on going conformity by 2797 BSI Group The Netherlands B.V. - Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands EU type examination (Module B) by 2777, SATRA TECHNOLOGY EUROPE LIMITED, Bracetown Business Park, Clonee, Dublin 15, D15 Y2NP, Ireland.

OTHER COMPLIANCE: - Medical Device CLASS I | Reg. EU 2017/745 - EN 455 1,2,3 & 4

Reg. CE 1935/2004 - Reg. UE 10/2011 - DM 21/03/1973 FOOD SAFE certified by: Chelab S.r.l., a Merieux NutriScience company Via Fratta, 25, 31023 RESANA (TV) Italy EN 374-4: 2013 Degradation results indicate the change in puncture resistance of the gloves after exposure to the challenge chemical.

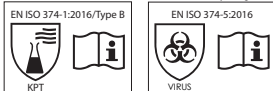EN Declaration of Conformity on www.reflexx.com

Disposable examination nitrile glove ■ Powder free, non-sterile ■ Shape: ambidextrous; five fingers ■ Sizes: 6-10 ■ Please read this USER INFORMATION SHEET carefully before using the glove.

RECOMMENDATIONS OF USE

Multi-purpose glove to protect the hand and/or the handled product, can also be used for general uses... This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals... When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties...

WARNINGS

Before using the product, try it in the real condition of use... The glove is a device subject to wear, so frequent replacement is recommended... Store the gloves in their original packaging in a dry place...

STORAGE AND DISPOSAL

Store the gloves in their original packaging in a dry place... Do not expose to direct sunlight and heat sources... The expiry date is 3 years from the date of manufacture indicated on the sales package...

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